

HealthCraft™ Easy Mount Grab Bars

Beautifully designed HealthCraft Grab Bars can be installed anywhere in the home to provide rock solid support.

Features and Benefits

▪ Easy Mount 9 Hole Flange

Unlike 3 hole flanges that limit screw placement, HealthCraft's innovative 9-hole Easy Mount flange design allows for greater mounting flexibility to structural studs.

Also compatible with specialized grab bar fasteners and anchors that mount directly behind drywall, tile and other surfaces should stud spacing not permit direct mounting. See website for details.

▪ No Rust

Stainless steel construction ensures that these grab bars will never rust. A simple wipe with a damp cloth is all that is required to maintain a clean attractive finish.


Powdercoated grab bars are an attractive option to enhance the already rust resistant properties of the stainless.

▪ Concealed Screws

Flange covers conceal the screws creating a more seamless and attractive look.

▪ ADA Compliant

All HealthCraft grab bars meet or exceed the guidelines of the Americans with Disabilities Act (ADA).

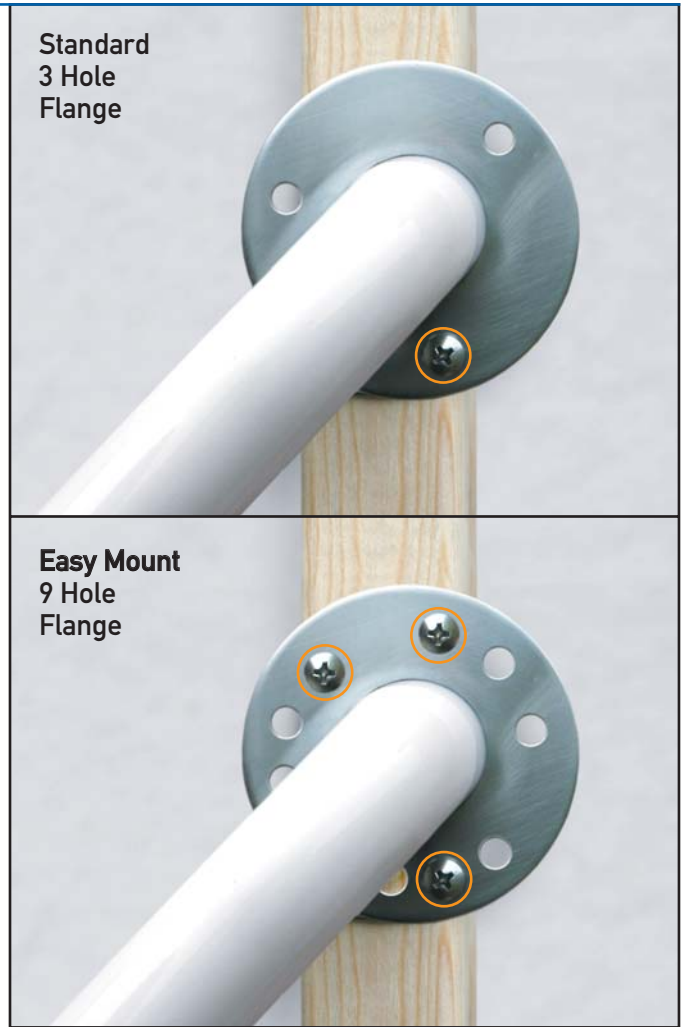

Technical Specifications:

Material	#304 Stainless Steel.
Finishes	White powder coat finish.
Rail Diameter	1.25" / 32mm (ADA compliant).
Rail Lengths	Measurements are from centre of flange plates: 18"/46 cm (accommodates standard stud spacing of 16" from center to center), 24"/61cm, 32"/81cm, 36"/91cm.
Wall Clearance	Distance between the rail and wall is 1.5" / 38mm.

Assistance Level



Made in China.

CAUTION: NOTE: Maximum user weight / load for grab bars is dependent on selection of appropriate mounting hardware for mounting surface. We have supplied mounting hardware for basic installs into wooden studs. HealthCraft recommends grab bar installation by a qualified contractor. All models are compatible with Winglts® Fasteners.