



SuperPole™ Floor Receptacle System

A portable assist pole that installs into a floor receptacle in a concrete floor. Best suited for institutions where room needs change frequently. Pole installs and removes from receptacle in seconds.

Features and Benefits

- **Floor Receptacle.** Installed in concrete, flush with floor surface to reduce trip hazards. When not in use, receptacle is sealed with a liquid tight cap.
- **Cap Removal / Storage.** Receptacle cap is unscrewed using pins located on the bottom of the pole. Prevent loss by storing the cap inside the top of the pole.
- **Installation.** Simply insert the pole into the receptacle and lock in place by turning the knob on top of the pole. Pole is secure and ready to use.



Technical Specifications:



F.R.S. Pole	57" / 145cm x 1.5" / 3.81cm x 1.5" / 3.81cm diameter. White powder coated steel.
F.R.S. Receptacle	3.9375" / 10cm x 1.875" / 4.8cm diameter.
Grip Length	20" / 51 cm. Washable, latex free grip.
Weight Capacity	450 lbs / 205 kg.



Unscrew receptacle cap in floor.



Remove cap and store in top knob of pole.



Insert F.R.S. pole into receptacle.



Fasten pole into receptacle by turning screw.