SuperPole™ Ultra

SP-ULTRA
(DC142 - RevB)

healthcraftproducts.com



SP400HDU


SP-EU


SP403HDU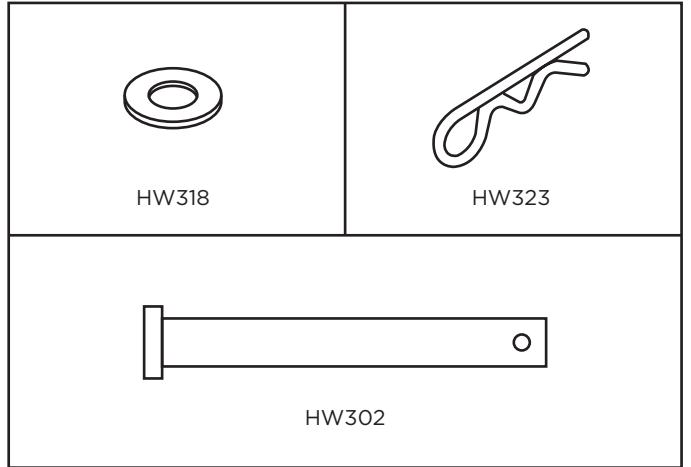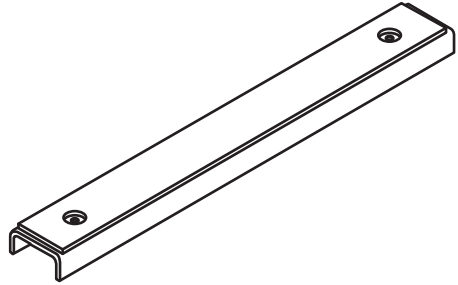

SP402

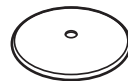
B

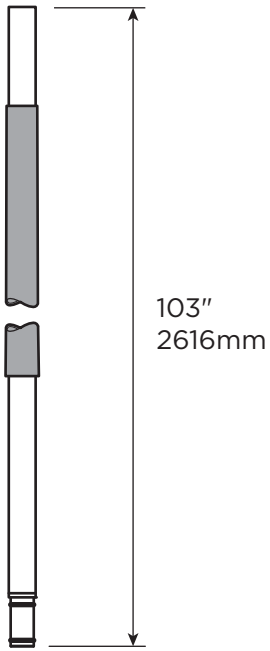
○ ← Ø1.50" / 38.1mm

FE / PVC / NBR
14lbs
6.3kg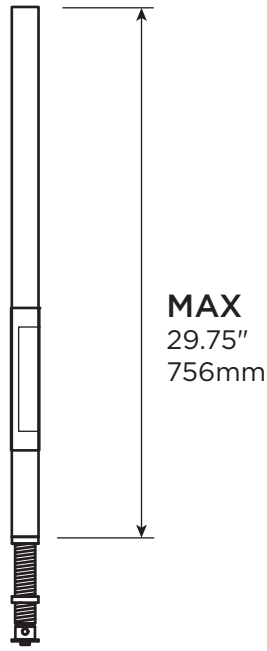

○ ← Ø1.50" / 38.1mm


FE / PVC / UHMW
8lbs
3.6kg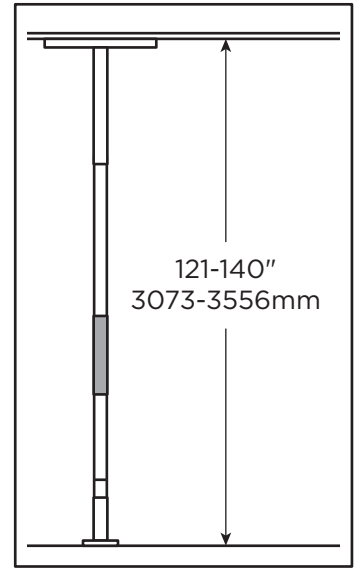


MAX
300lbs
136kg

C

1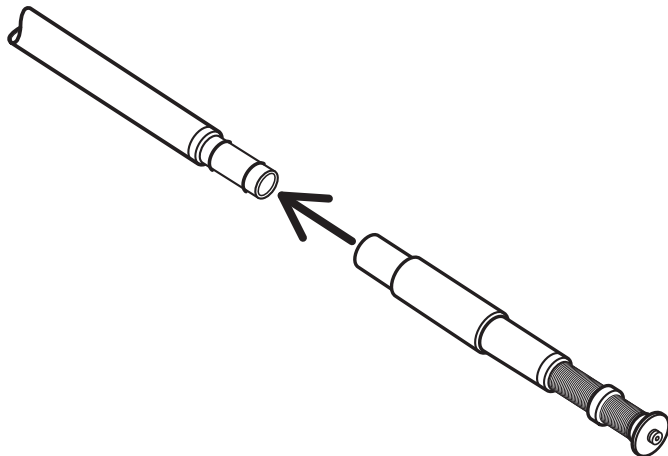

2

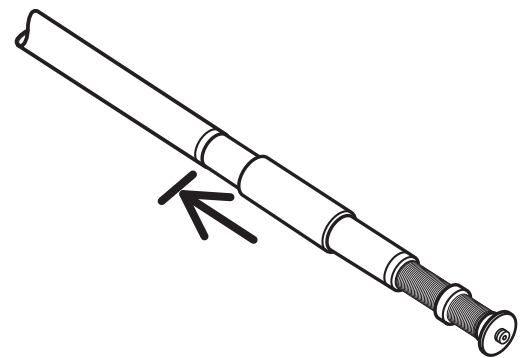OVERVIEW

Thank you for choosing the SuperPole Ultra from HealthCraft. Please read and understand the instructions in this manual (and the SuperPole Manual DC100); keep manuals for future reference.

The SuperPole Ultra is a high quality, commercial grade floor to ceiling safety pole (intended usage same as SuperPole). The SuperPole Ultra is intended for applications with floor to ceiling ranges from 121-140" / 3073-3556mm. The SuperPole Ultra is not to be used with the SuperBar, SuperTrapeze, Angled Ceiling Plate or Uni-Fit Extender. The SuperPole Ultra parts (pole, extension tube and top plate - labelled for convenience) should not be used with the components of any other floor to ceiling poles. The SuperPole Ultra is not to be used in any other way than described above. The SuperPole Ultra is covered under SuperPole warranty (DC100). Specifications subject to change without notice.

DELIVERY CONTENTS

FIGURE A.

1. SP400HDU - Ultra Pole
2. SP-EU - Custom Cut Extension Tube
3. SP403HDU - Ultra Top Plate
4. SP402 - Base Plate

TECHNICAL DATA

See Figure B.

INSTALLATION INSTRUCTIONS

FIGURE C.

1. Insert the end of the Ultra pole with the o-rings into the custom extension tube.
2. To ensure a snug fit, ensure there is no gap between the bottom shoulder of the Ultra pole and the extension tube. If required, use a hammer to tap the poles together (note: Use a wood block in between the hammer and extension screw to prevent damage).
3. Continue with SuperPole instructions (DC100).

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