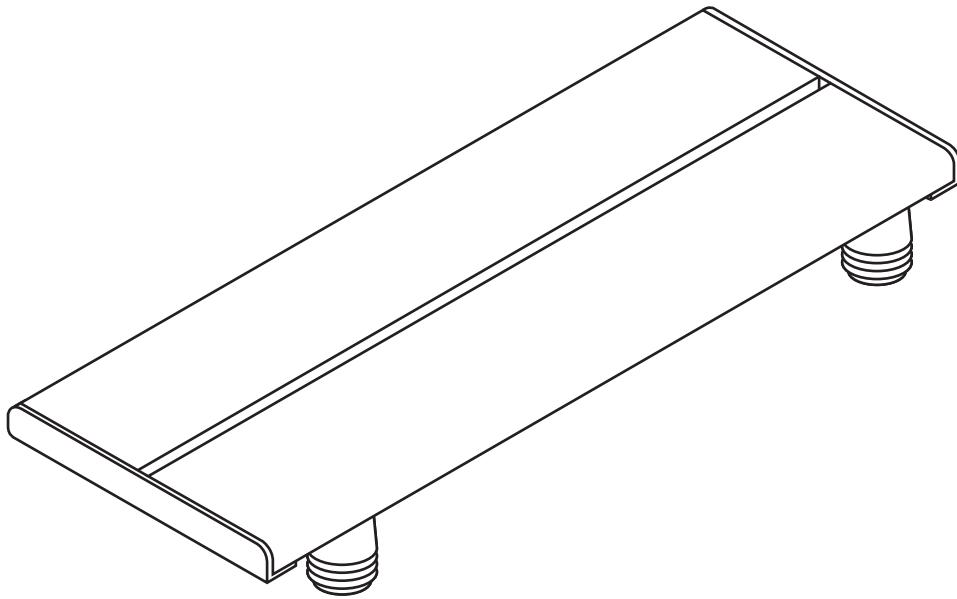
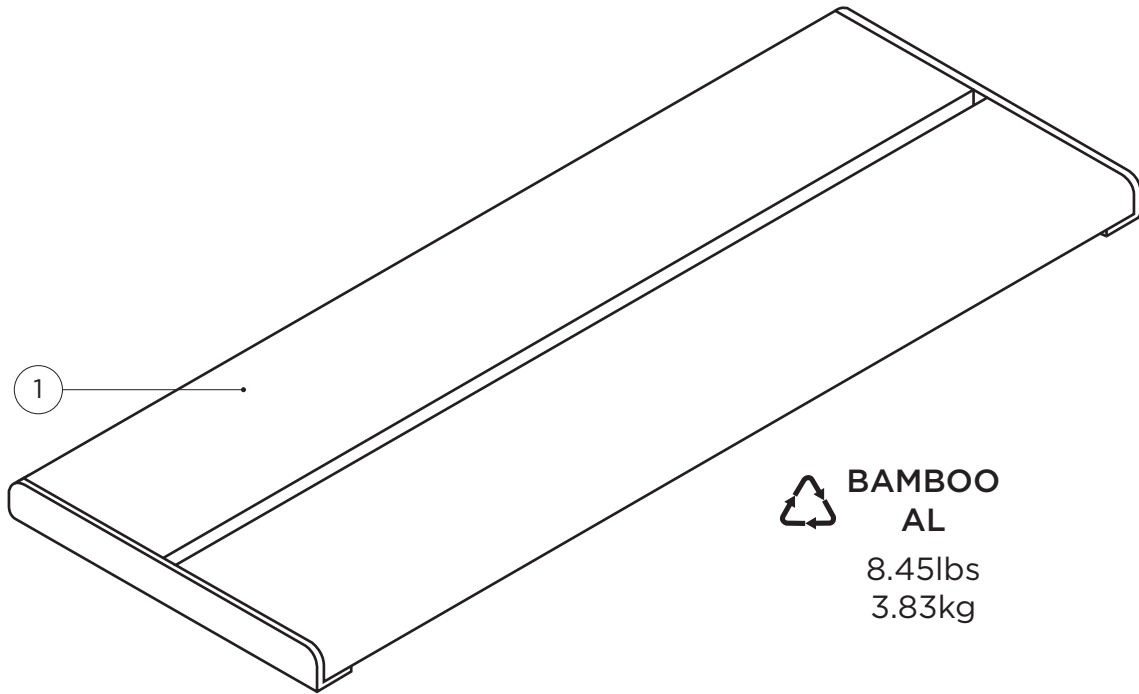
INVISIA

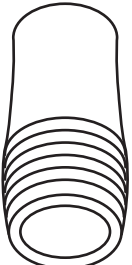
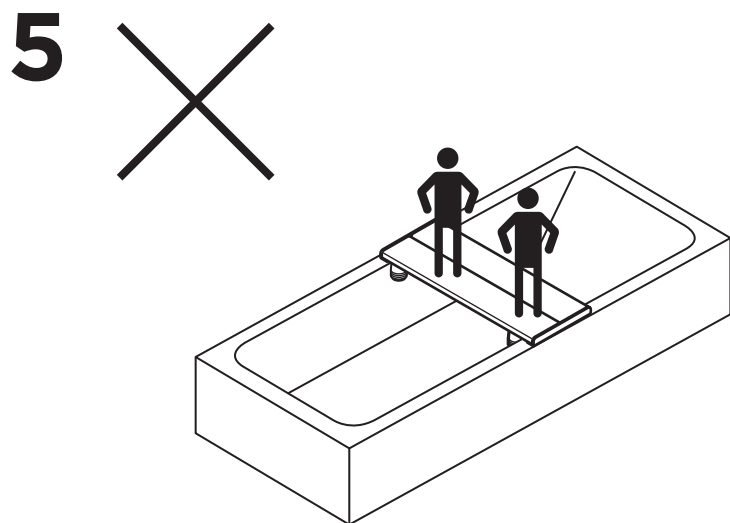
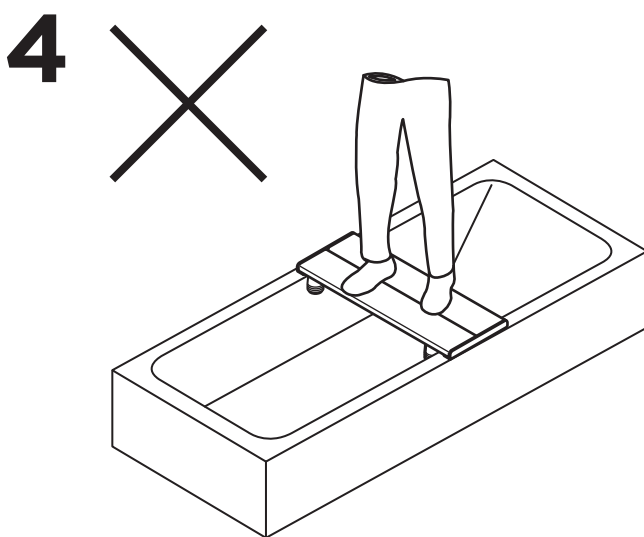
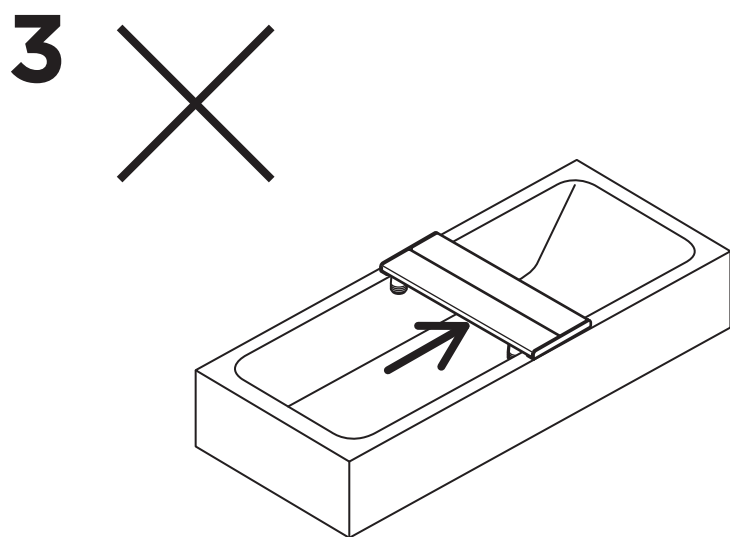
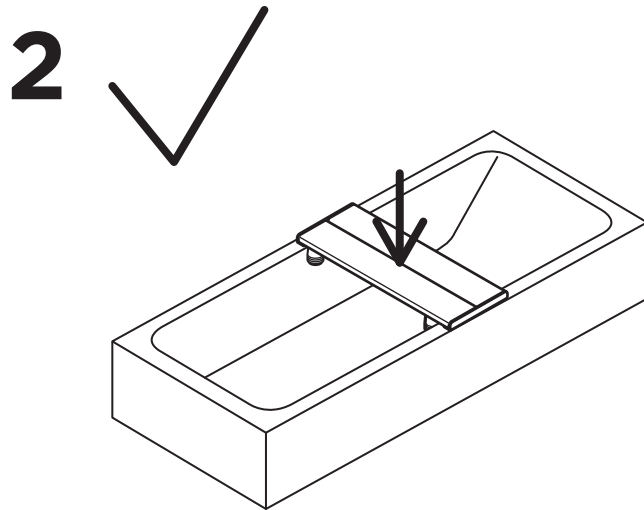
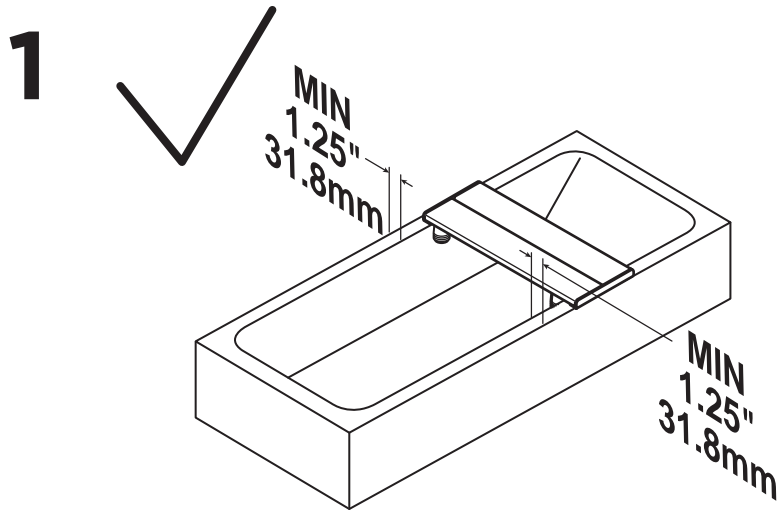
Bath Bench

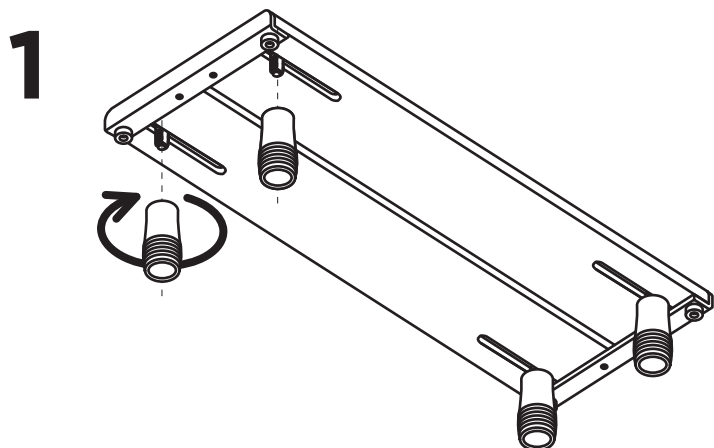
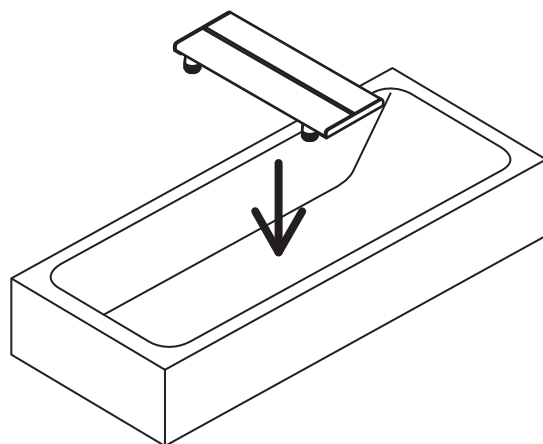
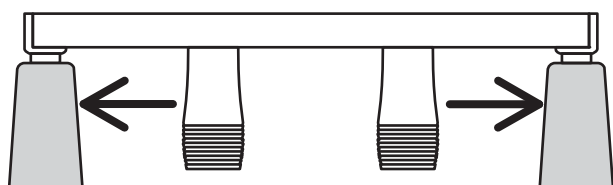
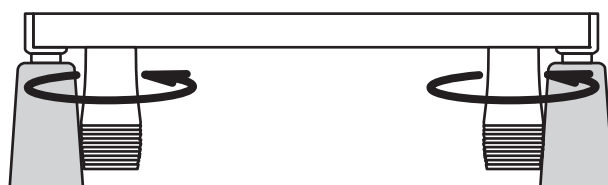
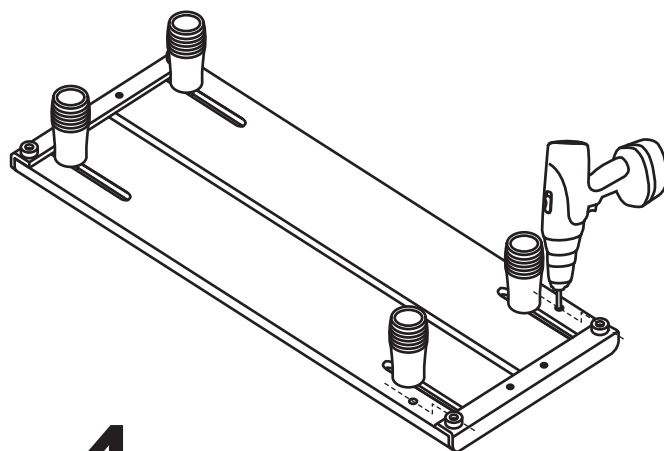
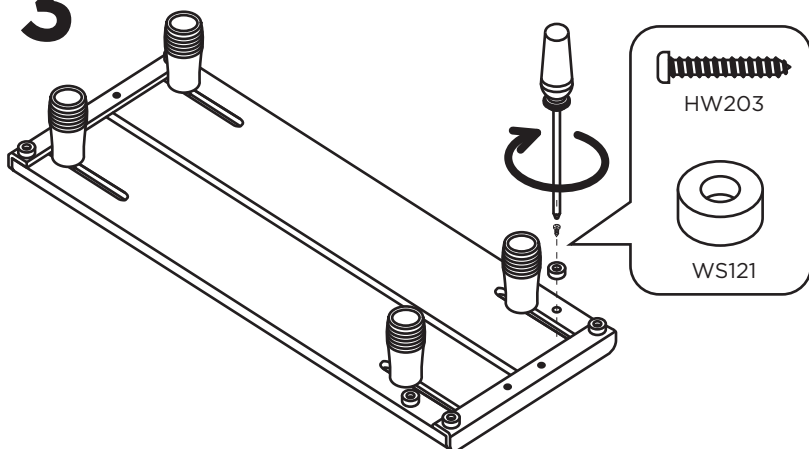
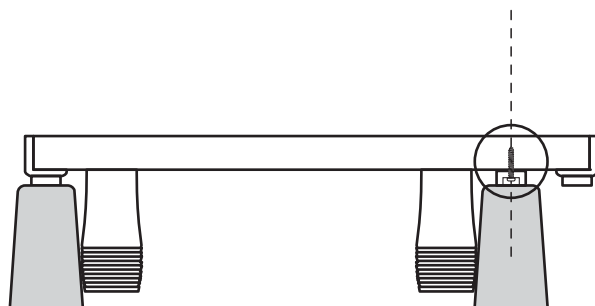
INV-WSBB
(DC148)

invisiacollection.com

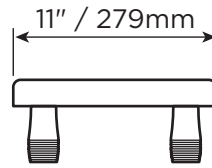
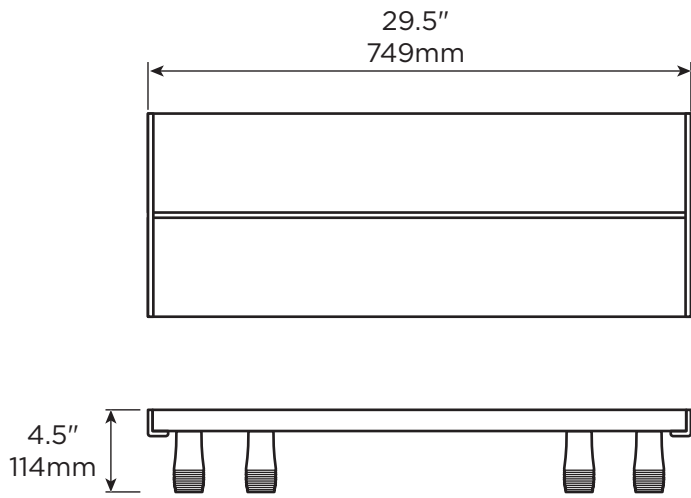
<p>2</p>  <p> PP + TPE 0.15lbs 0.07kg</p>	<p>3</p>  <p>WS121</p>
	<p>4</p>  <p>HW203</p>

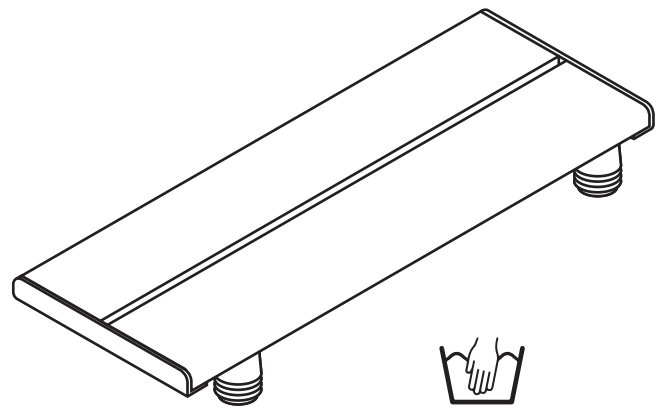
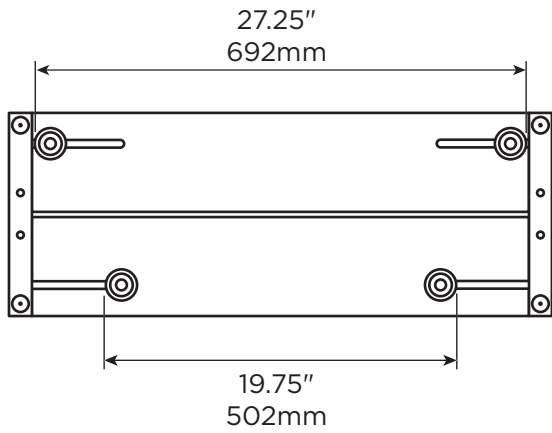
**2****3****4****1****2****3****4**

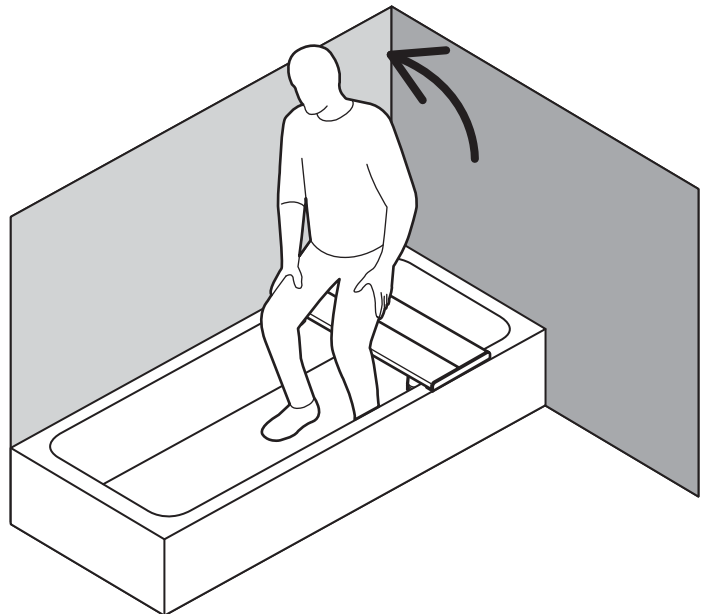
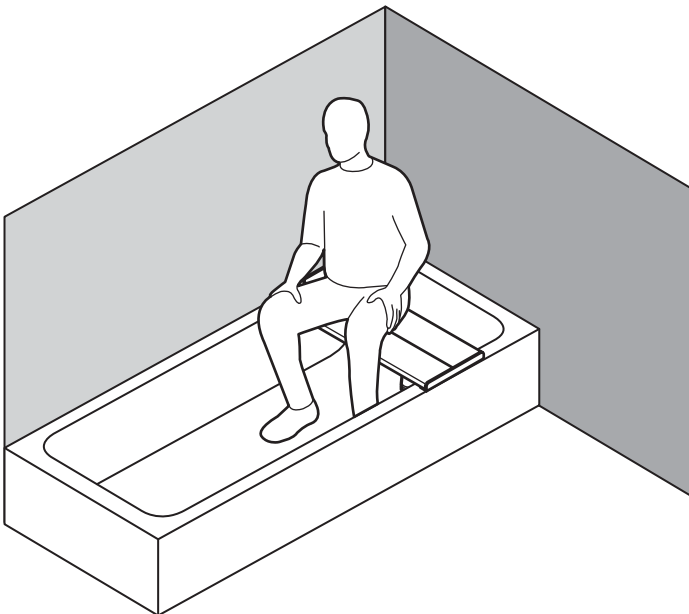
D

MAX
500lbs
227kg

E

English

OVERVIEW

Thank you for choosing the Invisia Bath Bench. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Invisia Bath Bench is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Invisia Bath Bench installed by a qualified contractor.

The Invisia Bath Bench is a high quality, commercial grade bath seat. The Invisia Bath Bench is intended to provide shower seating support for people with reduced mobility. The Invisia Bath Bench rests across the top surface of the bathtub rim, and rubber feet provide friction to reduce horizontal shifting. The maximum user weight for this product is 500lbs / 227kgs. Note: Product is not intended to support full body weight (i.e. do not stand on the seat). The Invisia Bath Bench is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described herein; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

HealthCraft Products Inc.
2790 Fenton Road
Ottawa, Canada
K1T 3T7

Gate 88
Kanalgatan 45B
Skellefteå, Sweden
93123

DELIVERY CONTENTS / BATH BENCH

FIGURE A.

1. Bath Bench
2. Plastic Friction Feet (x4)
3. Rubber Bumper (x2)
4. #6 x .75" Wood Screw, S/S (x2)

INSTALLATION WARNINGS

FIGURE B.

1. Bath Bench must rest on top of tub rim with a minimum horizontal flat surface of 1.25" / 31.8mm.
2. Bath Bench is intended to be used as a sitting surface only (i.e. downward vertical force only).
3. CAUTION - Do not exert sideward force on the bench.
4. CAUTION - Do not use bench for purposes other than sitting.
5. CAUTION - Do not use bench for more than 1 person at a time.
6. CAUTION - Seat and bathtub are slippery when wet.

INSTALLATION

FIGURE C.

1. Attach plastic friction feet to threaded sliders on the underside of the Bath Bench.
2. Rest Bath Bench on top of tub rim.
3. Slide plastic friction feet against the inside wall of the bathtub.
4. Tighten the plastic friction feet to secure. Inspect hardware regularly, and tighten if required.

BUMPER INSTALLATION: If bathtub width is more narrow than the Bath Bench, additional rubber bumpers may need to be installed.

1. Mark location of bathtub rim on bottom of Bath Bench.
2. Ensure bumpers are aligned with factory-installed bumpers. Drill pilot hole for #6 wood screws.
3. Secure rubber bumper to bottom of Bath Bench.
4. Inspect hardware regularly, and tighten if required.

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